Aesculap Endoscopic Technology

Bulldog Clips

Bulldog Clips

These bulldog clips, which can be applied laparoscopically, are used for intraoperative occlusion, and are therefore also called temporary clips.

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Particularly for laparoscopic vascular clips, easy and rapid clip removal is of the utmost importance. In partial nephrectomy, for instance, the blood supply to the entire kidney is interrupted during the resection procedure. In order to avoid damage to the kidney, the period of ischaemia should be kept as short as possible. For this reason, easy handling of the bulldog clips is extremely important.

Advantages of bulldog clips over laparoscopic clamps:

- The throw-off bulldog clip does not block any trocar access during application, since the application forceps can be withdrawn after the clip has been applied.
- The occluded vessel or organ can be manipulated as required and brought into a favourable position for the relevant stage of the operation. This causes less trauma.
- The bulldog clip always exerts the same defined pressure.

Special features of AESCULAP bulldog clips:

- Easy, safe and rapid clip application and removal
- Large portfolio with eight vascular clips and one intestinal clip
- The vascular clips are available as venous clips (reduced closing force) and arterial clips, as required
- The constant defined closing force is marked on each clip
- Vascular clips are provided with atraumatic DeBAKEY serrations
- Choice of application and removal forceps for straight or angled clip application

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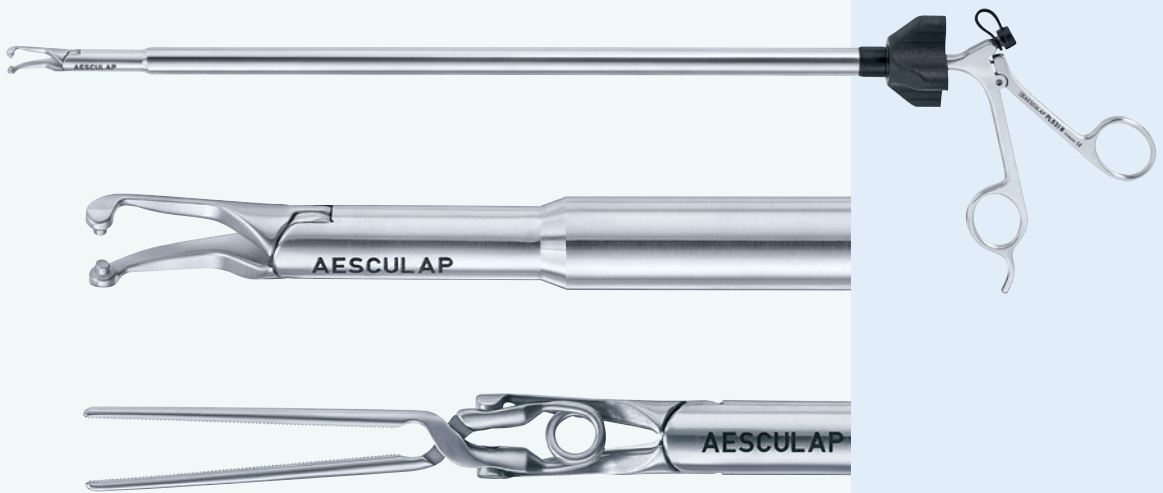
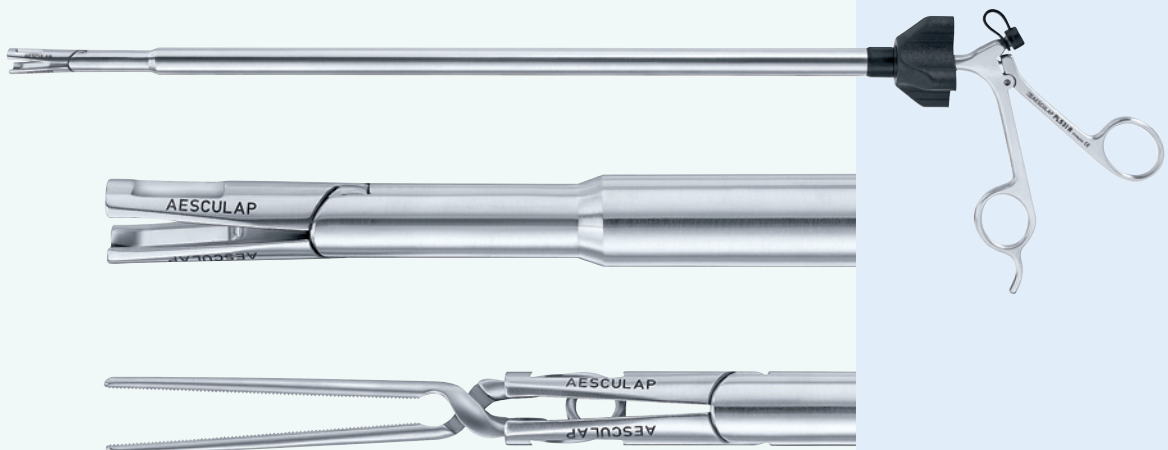
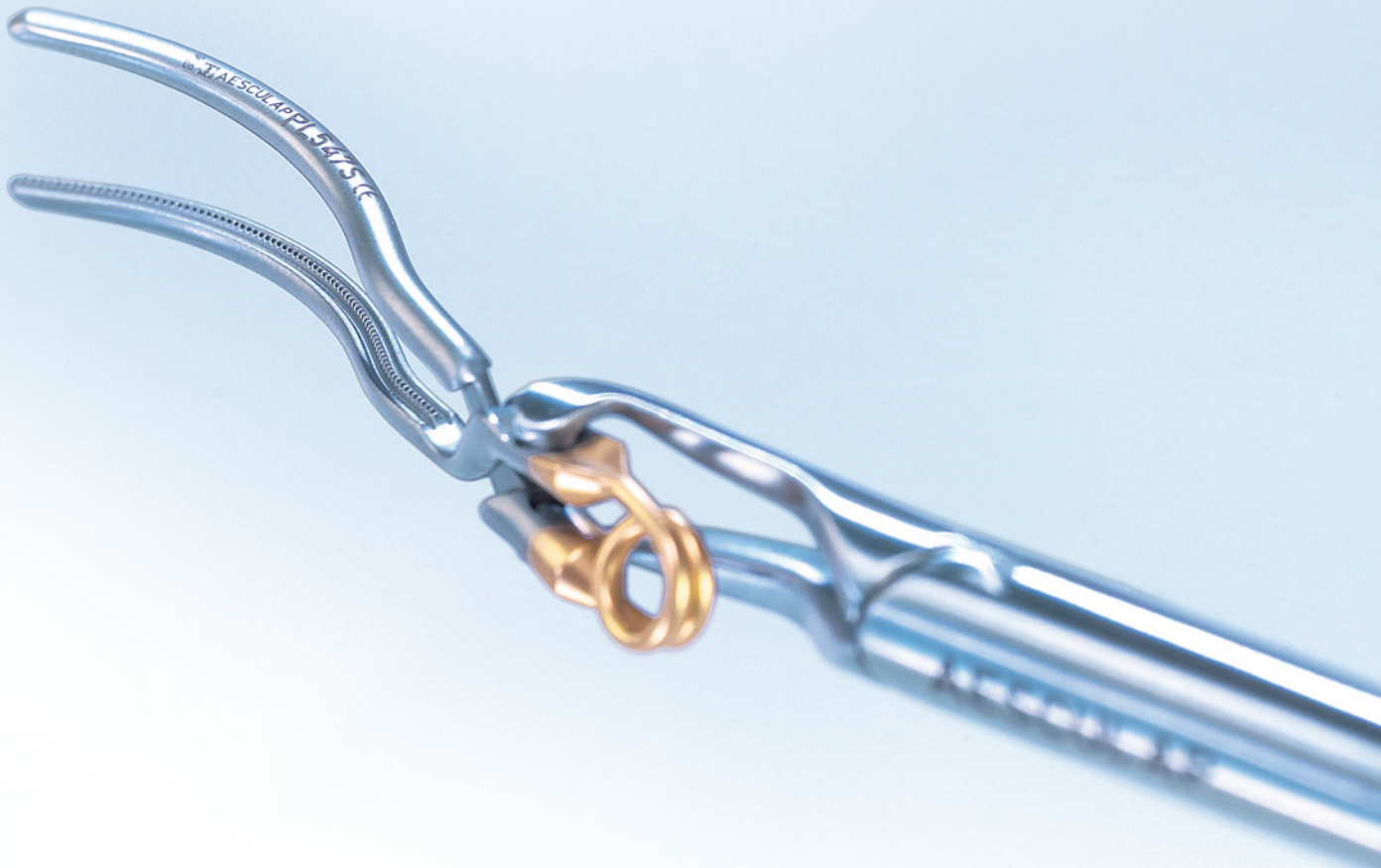
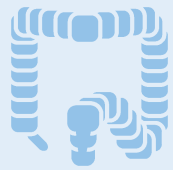
PL541S

Atraumatic endo intestinal clip acc. to Meyer, for temporary occlusion of the jejunum and colon. Two clips can be applied towards one another if necessary

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PL548S

PL549S

Venous endoscopic vessel clips with reduced closing force are especially marked at the end (gold).

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Phone +49 7461 95-0
Fax +49 7461 95-2600

www.aesculap.de

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.

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Aesculap AG & Co. KG