# Aesculap<sup>®</sup> Scarless Surgery

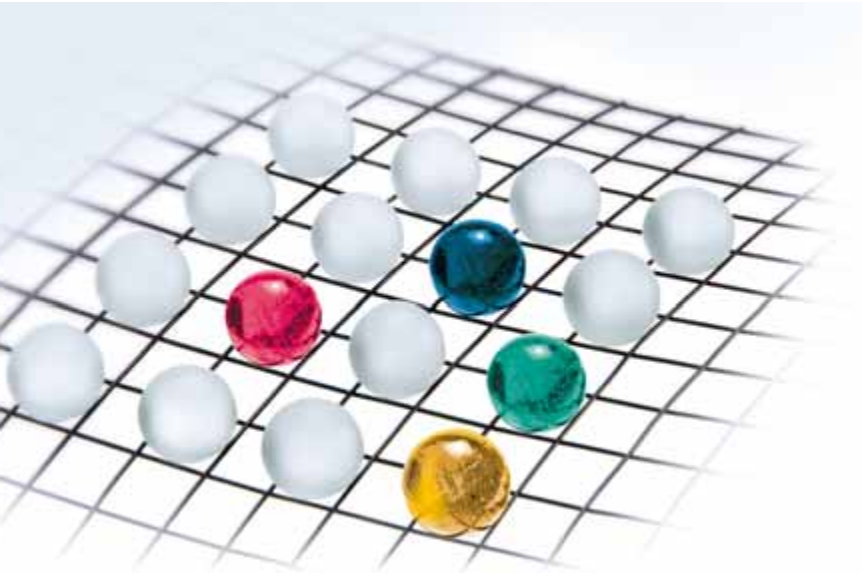
Solutions for less invasive surgical techniques

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Throughout the discussion about NOTES, related techniques and single-port procedures, the option to use instruments of smaller diameter has been paid little attention. This must surprise, as this alternative, in particular, allows applying the proven techniques of laparoscopic surgery, thus making the field of scarless surgery accessible to the majority of departments that perform minimally invasive surgery.

At our department we have been using the instrument set for mini laparoscopy (3.5 mm diameter) since April 2010. Initially the mini instruments were used rather selectively and as complementary equipment, but since then many laparoscopic procedures have been carried out mainly with mini instruments. The standard indications were laparoscopic cholecystectomy, appendectomy and hernia surgery, both in the TEP and the TAPP approach. For more complex procedures, such as sigmoid resection and fundoplication, the instruments are normally used in a supplementary function.

For the trocars, the common standard positions are applied, comparable to operating with 5-mm instruments. The incisions for the respective trocars, however, are much smaller. This becomes noticeable by the progress of postoperative healing, especially the fact that any scars are hardly visible anymore only a few days after the operation. E.g. for cholecystectomies, 10-mm or 5-mm endoscope trocars are used in the umbilical area, in combination with 3.5-mm trocars for the instruments.

Handling and operation of the AdTec mini instruments is easy and intuitive. The instruments further convince by their excellent quality of design and finish. Regarding performance, the smaller instruments are absolutely equal to the 5-mm variants normally used for comparable applications. All laparoscopic functions, such as cutting, grasping, dissecting and monopolar hemostasis are carried out with excellent ease and perfection. These robust and reliable instruments definitely offer a practical alternative to single-incision surgery and will have their place in the laparoscopic techniques of the future.

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Trocar sleeves and pins  
Trokarhülsen und -dorne

**Standard Technique**  
No change in the OR Setup

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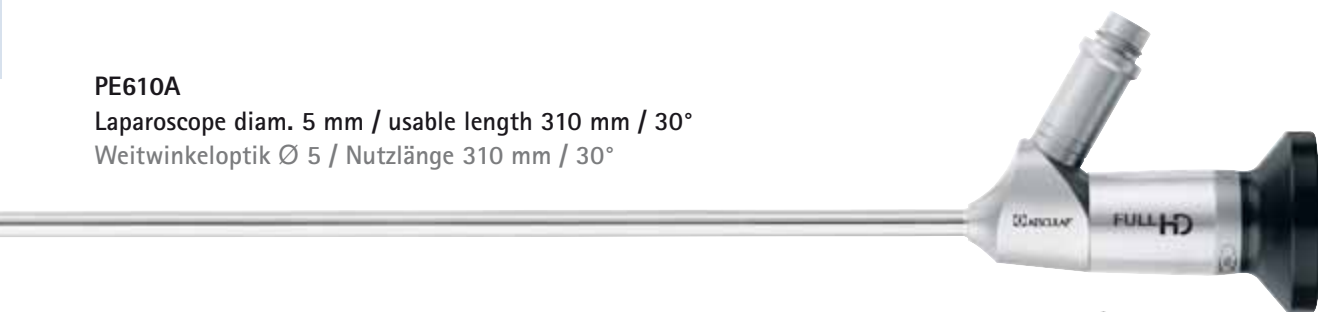
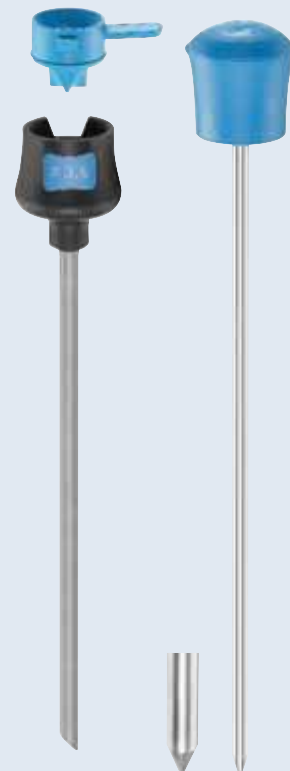
#### **EK380P**

Sealing unit for trocars diam. 3.5 mm  
20 units

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Options – single use  
Optionen – single use

**EK214SU**  
smooth, with stopcock diam. 5/110 mm  
**consisting of:**  
trocar sleeve, sealing unit, dilating trocar pin  
1 PAK of 6 pieces

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Single Use trocars in various diameters. Details can be found in the respective leaflets.

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Detailed information on the Aesculap® Reusable and Hybrid Trocar Solution product line can be found in our leaflets C621.

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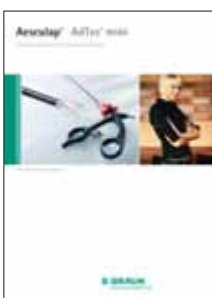
Detailed information on the Aesculap® Single Use Trocar Solution product line can be found in our leaflets C622.

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Detailed information on the Aesculap® AdTec® mini product line can be found in our leaflet C28511.

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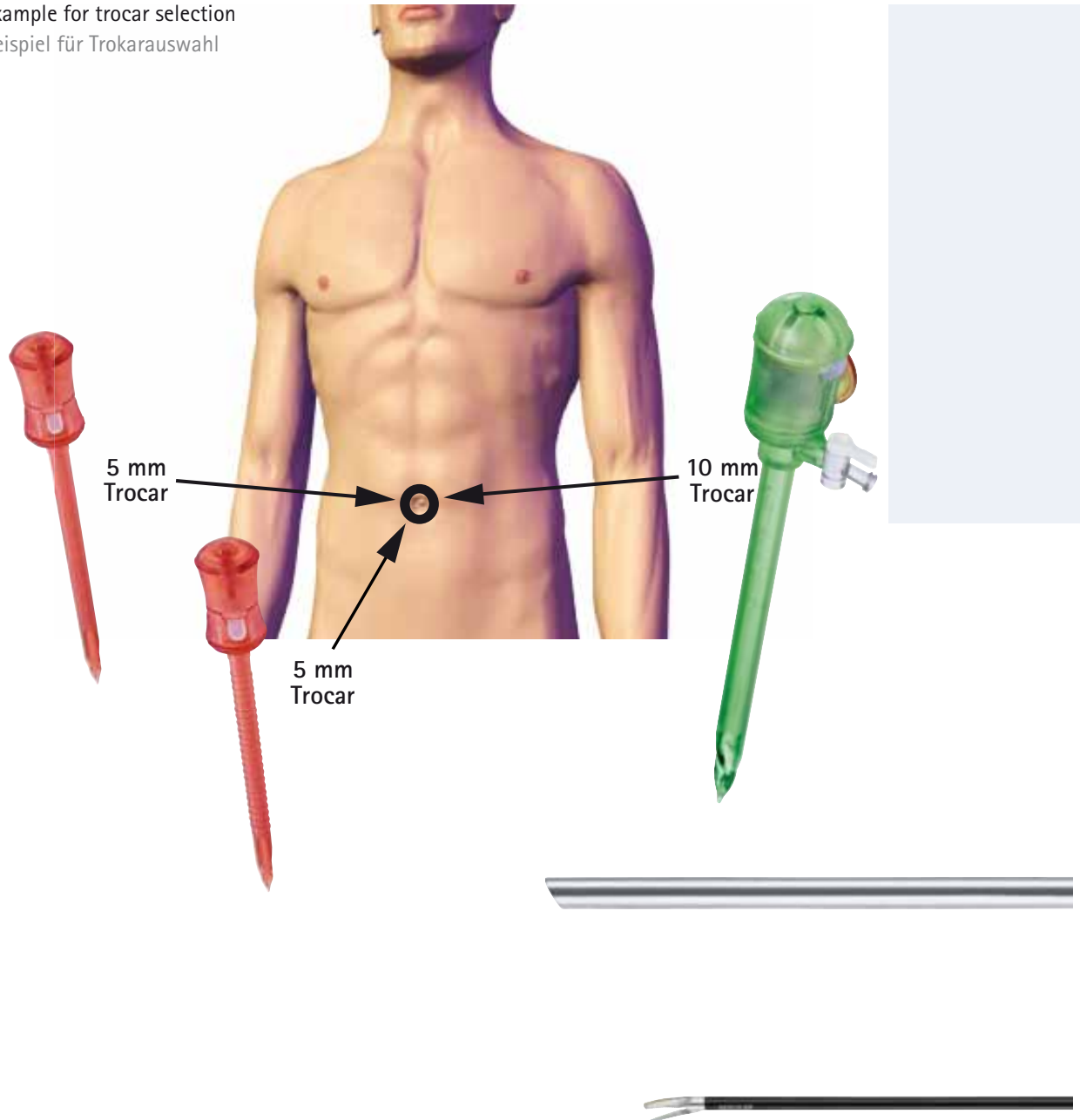
- Single trocars are positioned in the umbilical area
- Individual positioning of each trocar for a higher degree of freedom to manipulate the instruments and endoscope
- Low space requirement thanks to the spacesaving, slender trocar design.
- Low instrument friction for an easy manipulation of the instruments
- Low weight

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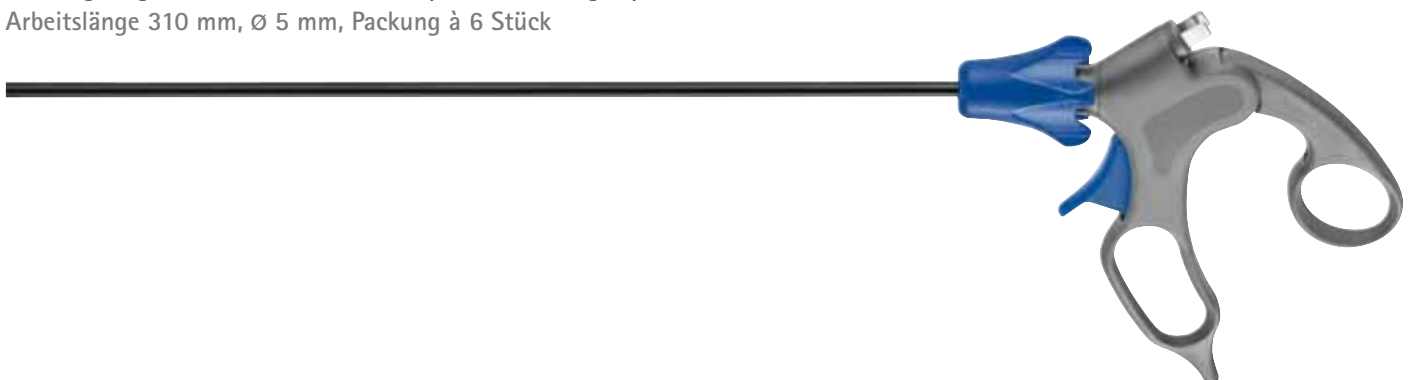
Länge 330 mm, Ø 10 mm, 30°

**PO800SU**

Working length 310 mm, diam. 5 mm, pack containing 6 pieces

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### Trocar sleeves and pins / Trokarhülsen und -dorne

#### Convenient

A system that is especially easy to handle and meets all requirements towards a trocar system.

Supporting the surgeon.

#### Safe

Patient and user safety are essential requirements towards the new trocar system.

For the patient.

#### Functional

Sterile supply, X-ray transparency and low weight are key characteristics of the system's high functionality.

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Zur Zufriedenheit aller.

Detailed information on the Aesculap® single use trocar systems can be found in our leaflet C622.

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**Single-use trocar Set**

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**EK214SU**

**consisting of:**  
trocar sleeve,  
sealing unit and  
dilating trocar  
pin

smooth,  
with stopcock,  
diam. 5 mm,  
length 110 mm  
PAK of 6 pieces

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**EK216SU**

**consisting of:**  
trocar sleeve,  
sealing unit and  
dilating trocar  
pin

threaded,  
with stopcock,  
diam. 5 mm,  
length 110 mm  
PAK of 6 pieces

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Einmal Trokarhülse

**EK192SU**

**consisting of:**  
Trocar sleeve and  
10 mm sealing  
unit with 5 mm  
flap converter

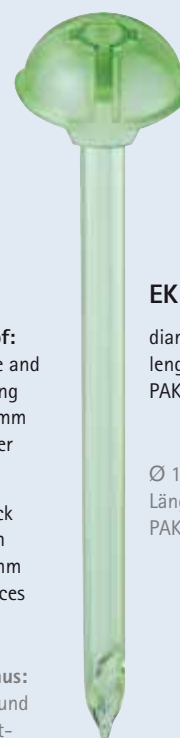
smooth,  
with stopcock  
diam. 10 mm  
length 110 mm  
PAK of 6 pieces

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diam. 10 mm  
length 110 mm  
PAK of 6 pieces

Ø 10 mm  
Länge 110 mm  
PAK à 6 Stück

# Single incision multiport laparoscopic technique

## Laparoskopische Multi-Port Technik

C 766  
AdTec Monopolar  
Instruments

C 240  
AdTec Combi

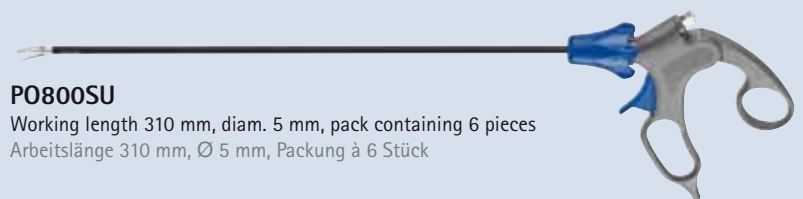
C 758  
AdTec Bipolar Instruments

For detailed information please ask for the following leaflets.

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### Operative procedure

- Application of the pneumoperitoneum via umbilical access and insertion of a 5 mm trocar
- Inspection of the abdominal cavity and exposure of the Pouch of Douglas with a 5 mm laparoscope
- Insertion of a 5 mm trocar under visual control in the posterior vaginal fornix and insertion of a long dissector to fixate the fundus of the gallbladder
- Along this insertion, an extra long 10 mm trocar is inserted for the long 45° / 10mm laparoscope
- Change of camera from the navel to the transvaginally inserted 10 mm laparoscope
- Insertion of a further 5 mm instrument to dissect the Calot's triangle through the umbilical incision
- Dissection and exposure of the cystic duct and the cystic artery, and subsequent closure using ligature clips
- Preparation and removal of the gall bladder, e.g. with a monopolar hook
- Removal of the gall bladder through the transvaginal access
- Closure of the posterior vaginal wall and of the umbilical access

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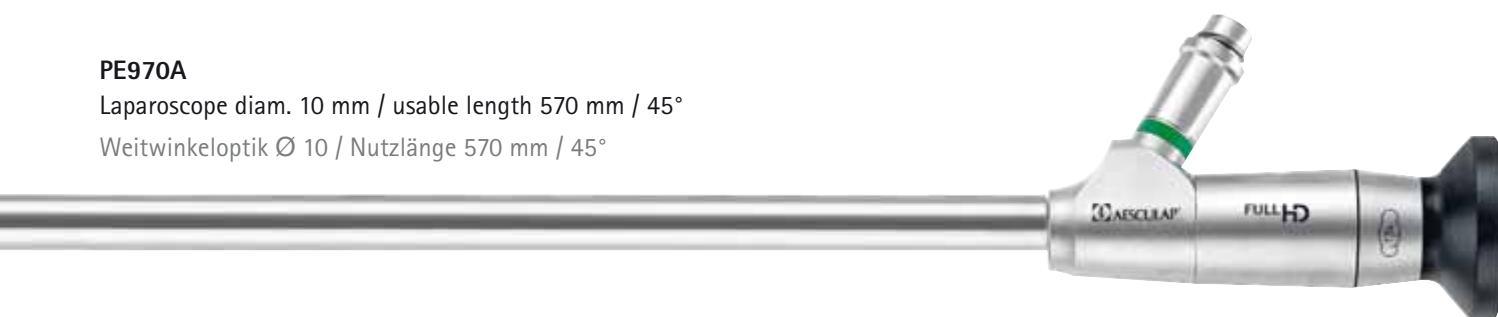
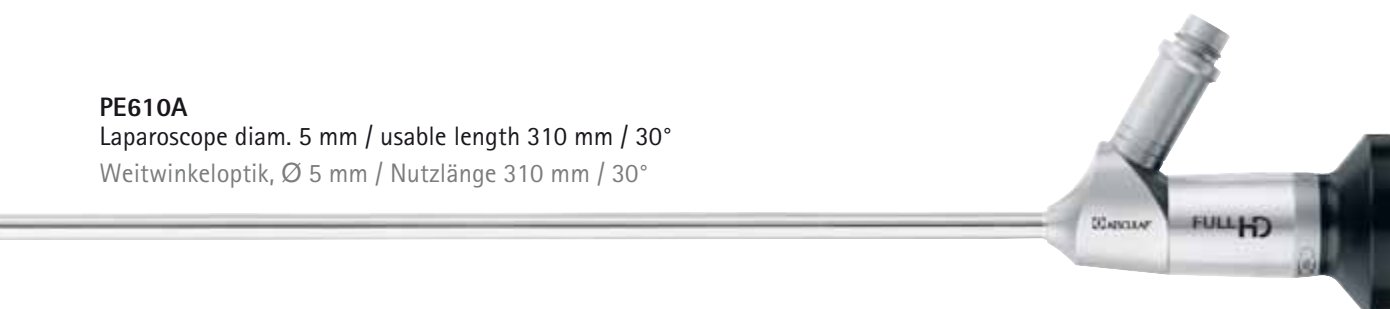
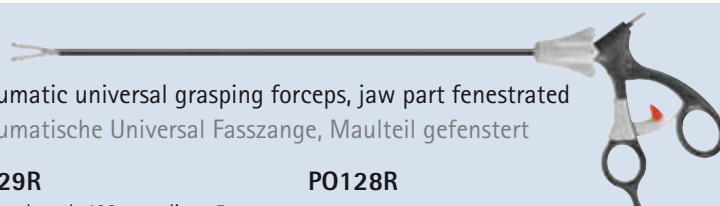
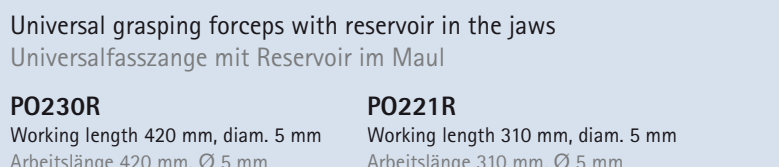
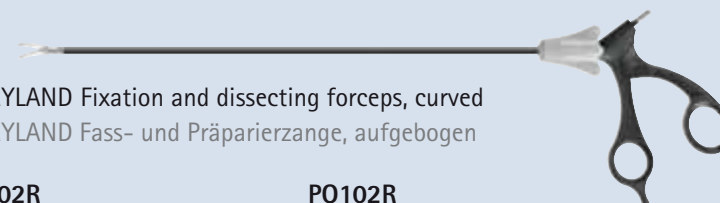
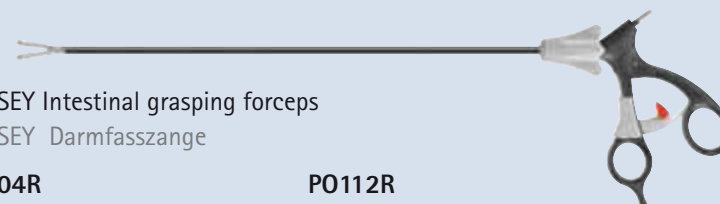
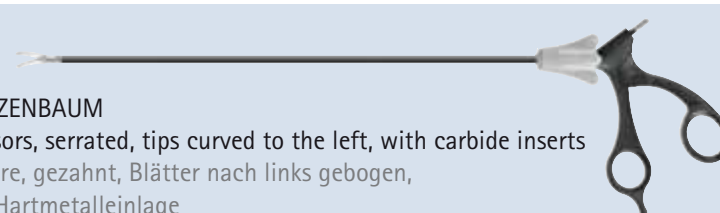
Ø 7 mm

EJ806P

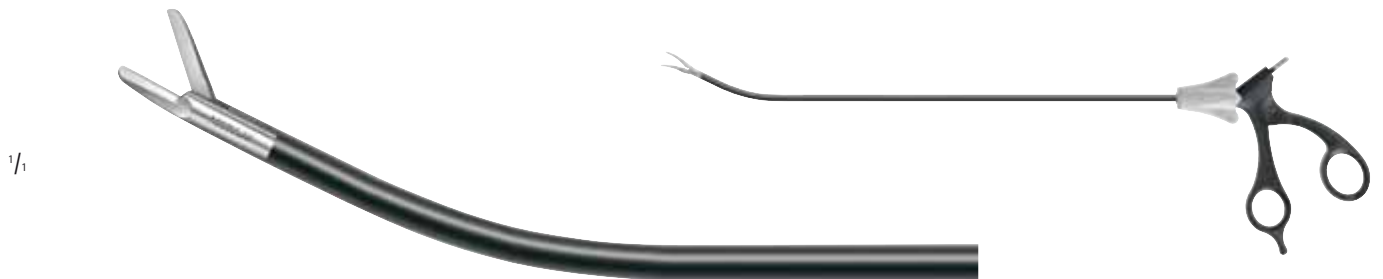
Trocar shaft  
as spare part  
package with 3 pieces

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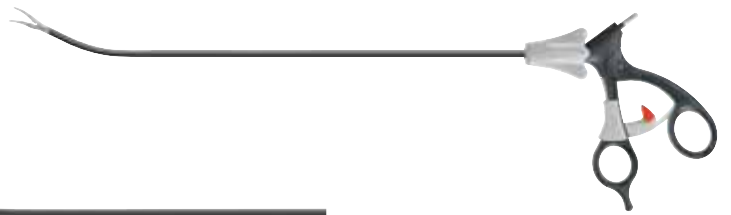
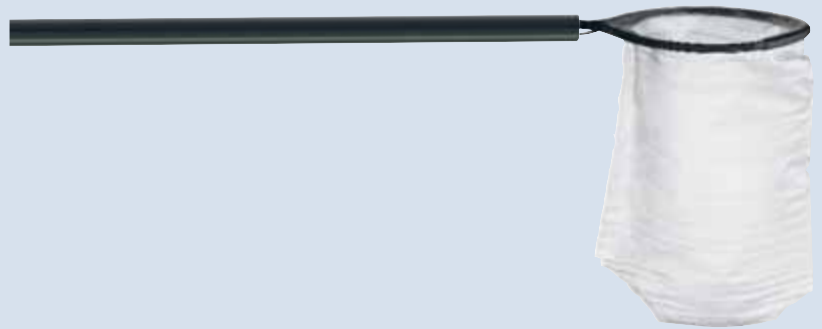
### Detachable retrieval bag · Abwerfbarer Bergebeutel

#### EJ023SU

Self-opening with metal ring  
Bag volume 260 ml

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#### EJ024SU

Self-opening with metal ring  
Bag volume 720 ml

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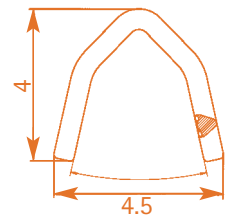
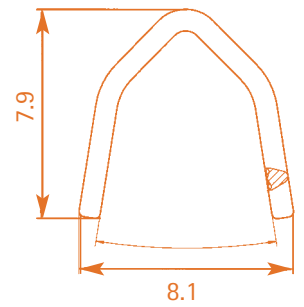
# Supplementary instruments Zusätzliche Instrumente

## Challenger® Ti-P

### First pneumatic driven multifire clip system

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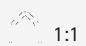
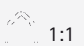
**C 094**  
AdTec® Bipolar  
Single Use Instruments

**C 623**  
Challenger Ti-P

**C 468**  
Clip Applicators for  
Titanium Clips

**C 357**  
Multi Interchangeable  
Trocar System

**C 621**  
Reusable and Hybrid  
Trocar Solution

**C 622**  
Single Use  
Trocar Solution

**C 267**  
Single Use Products

**C 413**  
Endofreeze

**C 730**  
Herloon

**C 829**  
Endoscopic Thyroid  
Surgery

**C 221**  
Endo-Rack

**C 232**  
Retrieval Bags

**C 467**  
Endoscopy Units and  
Accessories for Minimal  
Invasive Surgery

**C 101**  
Eddy DVD

**C 241**  
HD Camera

**C 629**  
Multi Flow Pump

**C 304**  
Electrosurgical  
Accessories

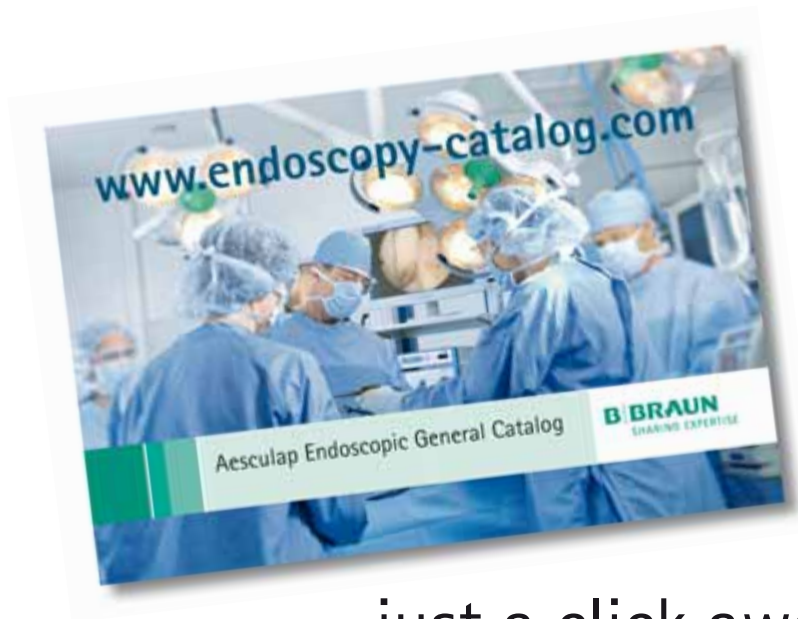
**C 771**  
Nelson deluxe  
Electrosurgery Unit

**C 306**  
GN 300  
Electrosurgery Unit

**C 725**  
Modular Monopolar  
HF-Electrodes

...just a click away!

[www.endoscopy-catalog.com](http://www.endoscopy-catalog.com)

[www.endoskopie-katalog.de](http://www.endoskopie-katalog.de)

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 (0) 74 61 95-0 | Fax +49 (0) 74 61 95-26 00 | [www.aesculap.com](http://www.aesculap.com)

Aesculap – a B. Braun company

The main product trademark 'Aesculap' and the product trademarks 'AdTec' and 'Durogrip' are registered trademarks of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

Brochure No. C62611

0711/0.75/2